

# INVACARE® SLINGS



Yes, you can:



Invacare offers a complete line of slings designed to meet the dynamic needs in both the long-term care and homecare settings. **Invacare® Reliant™ Series Slings** are an industry standard and deliver the ease of use, comfort and security that is standard in all Invacare Products. The **Invacare® Premier Series Slings** offer additional fabric and style options to meet a wider array of needs, as well as value-adding features for the caregiver and patient alike.

**Invacare® Reliant™ Slings** come in an array of styles to meet a variety of client needs. Reliant slings feature a unique, fully padded fabric that forms a soft yet supportive surface. The soft, brushed lining helps prevent sliding for ultimate comfort and skin protection, and the stretch-resistant feature helps the slings hold their shape for superior support and ease of positioning. For clients and caregivers alike, Invacare Reliant slings are designed with comfort and safety in mind.

### Features

Sling size and fit can vary significantly depending on patient weight and girth. These are general guidelines. Consult physician before sling selection.

All slings are constructed of polyester fabrics which can be used in wet or dry environments.


All Invacare® slings were designed specifically for use with Invacare® brand lifts for proper usage and optimal performance. To receive the maximum benefits from Invacare products, please see the size guide, product specs and operator manuals for solutions that best suit your needs.




1. **Invacare® Reliant™ Full-Body Slings**
2. **Invacare® Reliant™ Full-Body Slings with Commode Opening**
3. **Invacare® Reliant™ Divided Leg Slings**
4. **Invacare® Reliant™ Toileting Sling with Belt**

### General Sling Sizes\*

Color/Weight Chart

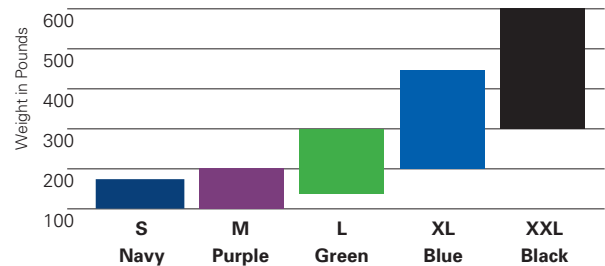

\*Color/weight chart is intended as a guide

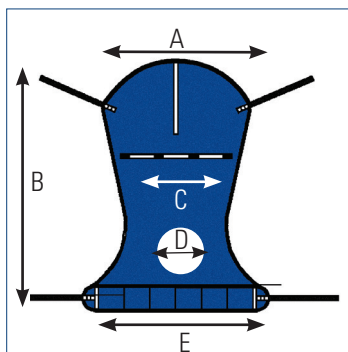


### Full Body Sling With/Without Commode Opening (CO)

An easy sling to use while transferring the patient from the bed to a wheelchair, toilet or shower chair or from floor to bed. It offers total head and neck support and the padded lower section provides additional under-thigh comfort and support. Use the full body sling for patients who are totally dependent, partially-dependent, non-weight bearing, very heavy or have limited head control. Use this sling for amputee patients (in a reclined position).

Compatible with the following lifts:

- Invacare® Reliant™ 450, 600 Lifts
- Invacare® Jasmine™ Lift
- Invacare® 9805P Lift
- Invacare® Birdie™ Evo XPLUS Lift

SIZE	A	B	C	D	E	Weight Capacity	Model #/Material
M	41.5"	53.3"	23.9"	N/A	43.7"	450 lb.	R110/Mesh
L	45.5"	58.3"	26.3"	N/A	47.7"	450 lb.	R111/Mesh
XXL	45"	55"	45"	N/A	45"	600 lb.	R140/Mesh
M	41.5"	53.3"	23.9"	6"	43.7"	450 lb.	R114/Mesh w/CO
L	45.5"	58.3"	26.3"	7"	47.7"	450 lb.	R115/Mesh w/CO
XL	45.5"	63.2"	26.3"	7"	47.7"	450 lb.	R116/Mesh w/CO
XXL	45"	55"	45"	7"	45"	600 lb.	R141/ Mesh w/CO
M	41.5"	54.7"	23.2"	N/A	41.8"	450 lb.	R112/Solid Composite Polyester Fabric
L	45.5"	60.3"	26.3"	N/A	45"	450 lb.	R113/Solid Composite Polyester Fabric
XL	45.5"	65.3"	26.1"	N/A	45"	450 lb.	R117/Solid Composite Polyester Fabric

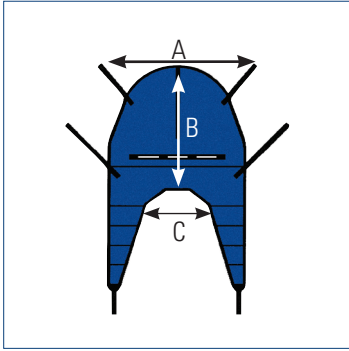


### Divided Leg Sling

The divided leg sling is a secure, easy-to fit general purpose sling, designed for the majority of sling users. Use the divided leg sling for patients who are totally dependent, partially dependent, non-weight bearing, very heavy or have limited head control. Use this sling when transferring the patient from the bed to a wheelchair, recliner or shower chair. The large opening also accommodates toileting.

Compatible with the following lifts:

- Invacare® Reliant™ 450, 600 Lifts
- Invacare® Jasmine™ Lift
- Invacare® 9805P Lift
- Invacare® Birdie™ Evo XPLUS Lift

SIZE	A	B	C	Weight Capacity	Model #/Material
S	33"	29.16"	12"	450 lb.	R100P/Solid Composite Polyester Fabric
M	37.5"	35.66"	17"	450 lb.	R100/Solid Composite Polyester Fabric
L	41"	40.66"	20.5"	450 lb.	R101/Solid Composite Polyester Fabric
XL	44.5"	43.16"	22.6"	450 lb.	R102/Solid Composite Polyester Fabric

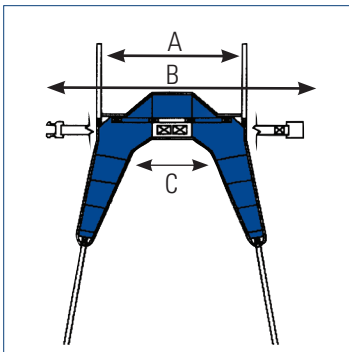


### Solid Fabric Toileting Sling

The Toileting Sling offers superb access for quick toileting and hygiene. It is easy to place and remove in a bed or chair situation. The fully padded design supports the back, offers under thigh comfort and allows for removal of clothing. Color coded positioning straps help align the sling properly on the hanger bar. **This sling requires the patient to have FULL head and neck control.**

Compatible with the following lifts:

- Invacare® Reliant™ 450, 600 Lifts
- Invacare® Jasmine™ Lift
- Invacare® 9805P Lift
- Invacare® Birdie™ Evo XPLUS Lift

SIZE	A	B	C	Waist Belt Min	Waist Belt Max	Weight Capacity	Model #/Material
L	36"	44.7"	20.5"	34"	66"	450 lb.	R121/Solid Composite Polyester Fabric

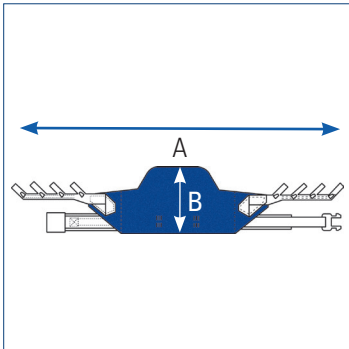


### Standing Sling (use with Stand-Up Lift)

Standing slings are for patients who are partially dependent, have at least 60% weight bearing capacity, have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance) and are able to bend at the hip, knees and ankles. Use the R130 or R131 standing slings for standing assistance, quick toileting, weight bearing practice and for transferring patient from bed to chair, or chair to bed.

Compatible with the following lifts:

- Invacare® Reliant™ 350 Lift
- Invacare® Rose™ Lift
- Invacare® Get-U-Up™ Lift
- Invacare® ISA™ Compact, XPLUS Lifts

SIZE	A	B	Waist Belt Min	Waist Belt Max	Weight Capacity	Model #/Material
One Size	60"	16"	27"	60"	440 lb.	R130/Solid Composite Polyester Fabric
XL	84"	16"	39"	84"	440 lb.	R131/Solid Composite Polyester Fabric

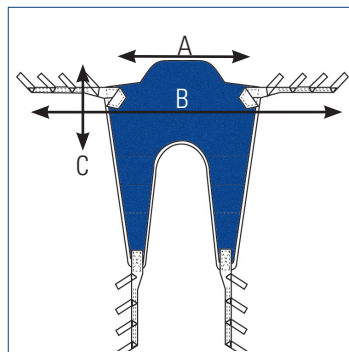

### Transfer Sling (use with Stand-Up Lift)


Transport slings are for transferring patients with minimal weight-bearing capacity, who have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance) and are able to bend at the hips, knees and ankles. Use the R134 and R136 transport slings for transferring patient from bed to chair, chair to car or general in-room transport. **Do not raise patient to a full standing position while using the transport sling.**

Compatible with the following lifts:

- Invacare® Reliant™ 350 Lift
- Invacare® Get-U-Up™ Lift
- Invacare® Roze™ Lift
- Invacare® ISA™ Compact, XPLUS Lifts

SIZE	A	B	C	Weight Capacity	Model #/Material
One Size	34"	13.5"	9"	440 lb.	R134/Solid Composite Polyester Fabric
XL	46"	13.5"	10"	440 lb.	R136/Solid Composite Polyester Fabric

**Invacare® Premier Series Slings** offer balanced solutions for transfers because they were developed in collaboration with Occupational and Physical therapists. Invacare Premier slings are available in a wide variety of styles and can be adjusted to ensure a comfortable and secure fit for the patient. Clearly labeled slings and easy attachment points make it simple for the caregiver to quickly attach the sling and lift the patient.

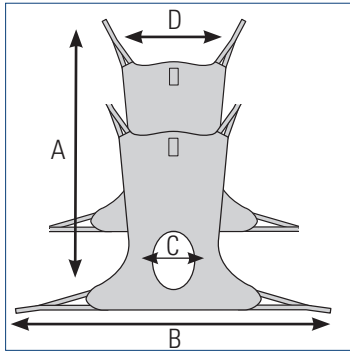


**Comfort Sling and Comfort Sling with Commode Opening**

Amputees and residents with chronic or postural pain have delicate needs that may require a constant seated position. Comfort Slings transfer them from a horizontal to seated position, with an additional model if toileting is also required.

Compatible with the following lifts:

- Invacare® Reliant™ 450, 600 Lifts
- Invacare® Jasmine™ Lift
- Invacare® 9805P Lift
- Invacare® Birdie™ Evo XPLUS Lift

SIZE	A	B	C	D	Weight Capacity	Model #/Material		
						Comfort Sling		Comfort Sling w/Toileting Opening
						Net Fabric	Spacer Fabric	Net Fabric
S	45.3"	41.4"	21.7"	40.2"	500 lb.	2485968	2485554	2451097
M	55.2"	46.1"	24.0"	46.5"	500 lb.	2485969	2485777	2451098
L	57.1"	50.8"	26.4"	50.4"	500 lb.	2485970	2485778	2451099
XL	57.5"	52.4"	29.6"	57.9"	550 lb.	2485947	2451090	2451100

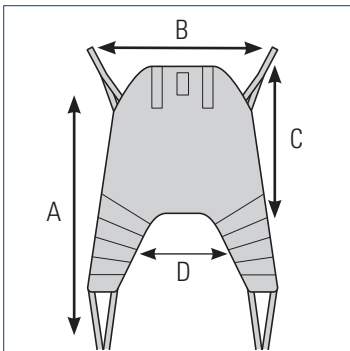


**Universal High Sling**

Supports the whole body and head in transfers to/from seated or lying position. Offers a comfortable recline for those with limited head, trunk or hip control. Universal High Sling allows arms to hang outside the sling.

Compatible with the following lifts:

- Invacare® Reliant™ 450, 600 Lifts
- Invacare® Jasmine™ Lift
- Invacare® 9805P Lift
- Invacare® Birdie™ Evo XPLUS Lift

SIZE	A	B	C	D	Weight Capacity	Model #/Material	
						Polyester Fabric	Spacer Fabric
S	42.6"	33.1"	28.8"	15.0"	500 lb.	2483777	2485956
M	55.9"	39.4"	32.7"	15.8"	500 lb.	2483778	2485957
L	61.1"	43.3"	36.2"	20.1"	500 lb.	2483779	2485958
XL	66.2"	45.3"	36.2"	21.7"	550 lb.	2485949	2485959

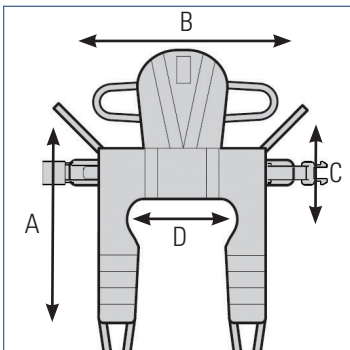


**Dress Toileting High Sling**

Featured belt offers secure support and positioning. Easy-to-apply Dress Toileting High Sling is suitable for seat-to-seat transfers and dressing or undressing during transfers to toilet. Dress Toileting High Sling is specially designed for users with reduced head control.

Compatible with the following lifts:

- Invacare® Reliant™ 450, 600 Lifts
- Invacare® Jasmine™ Lift
- Invacare® 9805P Lift
- Invacare® Birdie™ Evo XPLUS Lift

SIZE	A	B	C	D	Waist Belt Min	Waist Belt Max	Weight Capacity	Model #/Material
								Polyester Fabric
S	34.3"	28.0"	7.9"	16.9"	24.7"	46.7"	500 lb.	2478444
M	37.4"	31.5"	9.1"	19.3"	48.4"	55.1"	500 lb.	2478445
L	40.2"	35.5"	9.1"	23.6"	52.4"	59.1"	500 lb.	2478446
XL	43.3"	42.6"	10.6"	29.9"	72.4"	80.3"	550 lb.	2484780

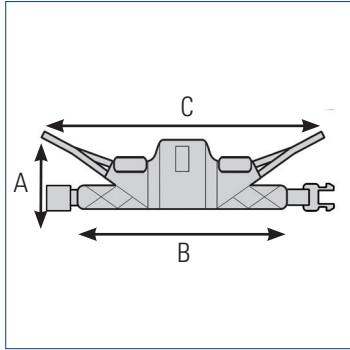


### Stand Assist Sling

Designed for use with the Invacare stand up lifts, the Stand Assist Sling offers a comfortable and safe transfer for those who have some weight bearing ability, but cannot stand independently. The sling is well-padded and features non-slip material on the side that touches the patient to assure a comfortable transfer position.

Compatible with the following lifts:

- Invacare® Roze™ Lift
- Invacare® Reliant™ 350 Lift
- Invacare® Get-U-Up™ Lift
- Invacare® ISA™ Compact, XPLUS Lifts

SIZE	A	B	C	Waist Belt Min	Waist Belt Max	Weight Capacity	Model #/Material
							Polyester Fabric
S	10.2"	33.1"	34.7"	24.7"	46.7"	450 lb.	2484685
M	13.0"	35.3"	46.1"	48.4"	55.1"	450 lb.	2484686
L	14.2"	37.4"	50.0"	52.4"	59.1"	450 lb.	2484687
XL	14.2"	40.2"	71.3"	72.4"	80.3"	450 lb.	2484688

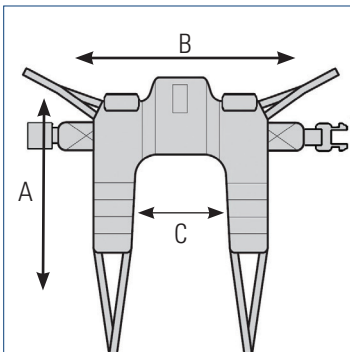


### Transfer Stand Assist Sling

Also designed for optimal use with the Invacare stand up lifts, this high-quality sling provides simple seated transfer for patients with minimal weight bearing ability.

Compatible with the following lifts:

- Invacare® Roze™ Lift
- Invacare® Reliant™ 350 Lift
- Invacare® Get-U-Up™ Lift
- Invacare® ISA™ Compact, XPLUS Lifts

SIZE	A	B	C	Waist Belt Min	Waist Belt Max	Weight Capacity	Model #/Material
							Polyester Fabric
S	33.9"	33.1"	9.5"	24.7"	46.7"	450 lb.	2484689
M	40.2"	35.3"	10.2"	48.4"	55.1"	450 lb.	2484690
L	46.5"	37.4"	13.0"	52.4"	59.1"	450 lb.	2484691
XL	48.5"	40.2"	13.8"	72.4"	80.3"	450 lb.	2485117

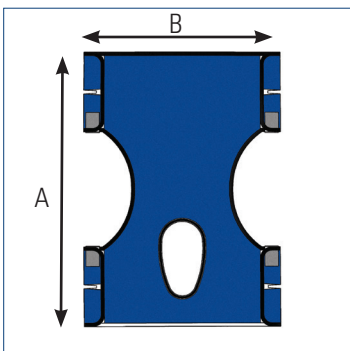

### Standard Four-Point Sling

Standard slings come with metal inserts where the chain or strap assembly is attached.













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













Compatible with the following lift:

- Invacare® 9805P Lift

SIZE	A	B	Weight Capacity	Model #/Material
One Size	40.5"	29"	450 lb.	9042/Solid Polyester without Commode
One Size	40.5"	29"	450 lb.	9043/Solid Polyester with Commode
One Size	40.5"	29"	450 lb.	9046/Mesh Polyester without Commode
One Size	40.5"	29"	450 lb.	9047/Mesh Polyester with Commode
<b>Chain or Strap Assembly Kit</b> (required for standard slings)				
One Size	N/A	N/A	450 lb.	9070 Sling Strap Assembly kit
One Size	N/A	N/A	450 lb.	9071 Sling Chain Assembly kit

## Slings for Floor Lifts













Compatible Lifts: RPL450-1 | RPL450-2 | RPL600-1 | RPL600-2 | BRDXPLUS | JASMINE | 9805P\*

Sling Style	Size	Color Code	Premier				Reliant				
			Style	Material		Weight Cap	Style	Material		Weight Cap	
			Universal High	Solid Poly	Net	Spacer	lb.	Divided Leg	Brushed Poly	Mesh	lb.
Divided Leg/ Universal	S			2483777	-	2485956	500		R100P	-	450
	M			2483778	-	2485957	500		R100	-	450
	L			2483779	-	2485958	500		R101	-	450
	XL			2485949	-	2485959	550		R102	-	450
			Comfort	Solid Poly	Net	Spacer	lb.	Full Body	Brushed Poly	Mesh	lb.
Full Body/ Comfort	S			-	2485554	2485554	500		-	-	500
	M			-	2485969	2485777	500		R112	R110	500
	L			-	2485970	2485778	500		R113	R111	500
	XL			-	2485947	2451090	550		R117	-	550
	XXL			-	-	-	-		-	R140	600
			Comfort w/CO	Solid Poly	Net	Spacer	lb.	Full Body w/CO	Brushed Poly	Mesh	lb.
Full Body/ Comfort with commode opening	S			-	2451097	-	500		-	-	500
	M			-	2451098	-	500		-	R114	500
	L			-	2451099	-	500		-	R115	500
	XL			-	2451100	-	550		-	R116	550
	XXL			-	-	-	-		-	R141	600
			Dress Toileting High	Solid Poly	Net	Spacer	lb.	Toileting	Brushed Poly	Mesh	lb.
Toileting/ Hygiene	S			2478444	-	-	500		-	-	-
	M			2478445	-	-	500		R121	-	450
	L			2478446	-	-	500		-	-	-
	XL			2484780	-	-	550		-	-	-

\*Standard Four-Point slings also are available for 9805P lift only. Details on page 5.

## Slings for Stand Assist Lifts

Compatible Lifts: RPS350-1 | RPS350-2 | ISACOMPACT | ISAXPLUS | ROZE | GHS350

Sling Style	Size	Color Code	Premier				Reliant				
			Style	Material		Weight Cap	Style	Material		Weight Cap	
			Stand Assist	Solid Poly	Net	Spacer	lb.	Standing	Brushed Poly	Mesh	lb.
Stand Assist	S			2484685	-	-	450		-	-	-
	M			2484686	-	-	450		-	-	-
	L			2484687	-	-	450		R130	-	440
	XL			2484688	-	-	450		R131	-	440
			Transfer Stand Assist	Solid Poly	Net	Spacer	lb.	Transfer	Brushed Poly	Mesh	lb.
Transfer	S			2484689	-	-	450		-	-	-
	M			2484690	-	-	450		-	-	-
	L			2484691	-	-	450		R134	-	440
	XL			2485117	-	-	450		R136	-	440

Invacare Corporation

©2022  
Invacare Corporation.  
All rights reserved.  
Trademarks are identified  
by the symbols ™ and  
©. All trademarks are  
owned by or licensed  
to Invacare Corporation  
unless otherwise noted.  
Specifications are subject  
to change without  
notification.  
Form No. 11-250  
Rev. 10/22  
221001